EAGLE RESPIRATOR

180021, 180022, 180023, 180024, 180025, 180026

EN Filtering half-mask

EAGLE FILTERS LTD
William Ruthin katu 1
FI-48600 KOTKA
FINLAND

+358 10 420 3070

respirators@eaglefilters.earth

www.eaglefilters.earth

CE 0161

CAT III

DECLARATION OF CONFORMITY WWW.EAGLEFILTERS.EARTH/CERTS

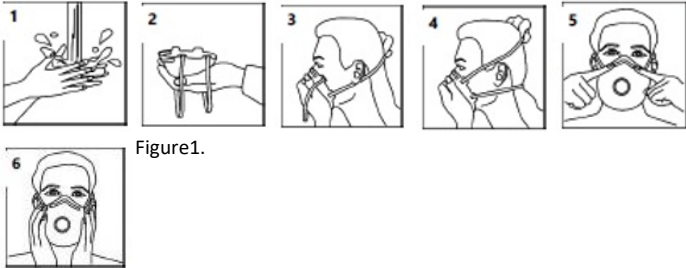
Figure1.

PICTOGRAMS

					NR
EN See information supplied by manufacturer	Temperature range of storing	Maximum relative humidity of storing	YYYY/MM End of shelf life	Do not store in direct sunlight	Not reusable

EN

This product has been manufactured in compliance with Regulations EN 149:2001+A1:2009, EU 2016/425 Respiratory Protective Devices - Filtering half masks to protect against particulates. Requirements, testing, marking, in compliance with certificate numbers 21/3729/00/0161 and 21/3696/00/0161 by AITEX, Plaza Emilio Sala nº 1, Alcoi, Spain, Notified Body 0161

OBTAINED VALUE: FFP1 for 180021 and 180022
FFP2 for 180023 and 180024
FFP3 for 180025 and 180026

Maximum penetration of the filter material:

Classification	95 l/min % Max.
FFP1	20
FFP2	6
FFP3	1

WARNING: THIS PARTICLE FILTERING HALF-MASK MUST NOT BE USED FOR MORE THAN ONE WORK SHIFT (NR).

DON'T USE IF DAMAGED.

CLOGGING TEST NOT EVALUATED.

NOT FLAME RESISTANT.

NOT TESTED FOR USE IN POTENTIALLY EXPLOSIVE ATMOSPHERES.

STORAGE: Keep the product away from unnecessary exposure to sunlight in dry humid places and protected against any aggressive agents.

PACKAGING: Plastic bag and cardboard box.

This PPE's woven materials nor attached components are not known to cause any irritation or inconvenience to the user.

FEATURES:

- adjustable nasal seal
- adjustable lower rubber band
- stiffened mask body
- exhale valve (optional)

DONNING, FITTING AND ENSURING TIGHTNESS OF THE FILTERING HALF-MASK (FIGURE 1.)

1. Wash your hands before you touch the mask.
2. Take the mask out of the package and place it on the palm of your hand, in a way that both rubber bands are hanging down and are not in turns.
3. Place the mask on your face and stretch the lower elastic band over your head to your neck. If you wear glasses, you may remove them while putting on the mask.
4. Stretch the upper elastic band behind your head to the point where the mask feels best to seal against your face. If the mask feels loose you can adjust the lower rubber band with the included pearl.
5. Place the mask in a position where it feels most comfortable and press evenly against face. push the nasal seal to the best position.
6. Before starting to work, make sure that the mask is tight to your face: cover the outside of the mask with your hands and inhale vigorously. If air is felt to flow under the edge of the respirator, adjust the nasal seal and correct the position of the mask until you feel the mask is tight. If you don't get the mask to seal on your face, do not go into a dangerous area. Do not use the mask if you have facial hair as the mask's effectiveness will reduce significantly.

The filtering half-mask does not protect:

- against gases or vapours
- by lack of oxygen (oxygen must be at least 19.5% by volume)
- by excessive oxygen content
- in situations where the impurities or the risk they can cause are unknown
- if it is not properly sealed by user

Leave your working area and go to safe area immediately if:

- you have difficulties to breath
- you feel dizzy or fatigue

The user of the filtering half-mask must be instructed in, how to put on the mask, how to ensure it's tightness, how to remove the mask and how to dispose it. Proper use of the mask at all stages of use is important. If the instructions are not followed, the user of the mask may be exposed to health hazards from which the mask is intended to be protecting.

INSPECTION BEFORE USE:

Wash or disinfect your hands before touching the mask. Check that the packaging of the mask is undamaged, that the mask has all its components such as the cup, rubber bands and optional valve and that they are intact. Check the expiration date of the mask and do not use if expired.

USE, REMOVAL AND DISPOSAL OF RESPIRATOR:

Use the mask continuously. Do not touch the outside of the mask during use. Do not temporarily remove it from the face during use. Mask needs to be changed to a new one if you need to remove it from your face or when damaged. Use the mask for a maximum of one working shift. It is recommended that the operating period does not exceed four hours.

Wash or disinfect your hands before removing the mask. Remove without touching the outside of the cup. Lift it off your face, for example by grasping the sides of the respirator. In accordance with food workplace practice, place the used mask either in a bag and close it in a rubbish bin provided for this purpose. Finally wash or disinfect your hands.

MAINTAENANCE, CLEANING AND DISINFECTION

Disposable non reusable.

STORAGE:

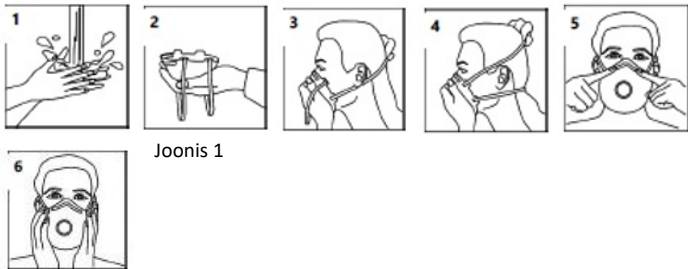
Store the respirators in a dry place with a temperature between -20 and +40 and a relative humidity below 80%

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