Saana

301 / 302 / 303 / 304 / City

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EN

This product has been manufactured in compliance with the harmonized standard EN 149:2001+A1:2009 and PPE regulation EU 2016/425, Respiratory Protective Devices - Filtering half masks to protect against particles. Requirements, testing and marking, in compliance with certificate number 0598/PPE/22/2224 issued by SGS Fimko Takomotie 8, FI-00380 Helsinki, Finland. Notified Body 0598

OBTAINED VALUE: FFP2 for Saana 301/ 302/ Saana City
FFP3 for Saana 303/304

Maximum penetration of the filter material:

Classification	95 l/min % Max.
FFP1	20
FFP2	6
FFP3	1

WARNING: THIS PARTICLE FILTERING HALF-MASK MUST NOT BE USED FOR MORE THAN ONE WORK SHIFT (NR).

DON'T USE IF DAMAGED.

CLOGGING TEST NOT EVALUATED.

MATERIALS NOT HIGHLY FLAMMABLE BUT NOT RECOMMENDABLE TO BE USED NEAR OPEN FIRE.
NOT TESTED FOR USE IN POTENTIALLY EXPLOSIVE ATMOSPHERES.

STORAGE: Keep the product away from unnecessary exposure to sunlight in dry humid places and protected against any aggressive agents.

PACKAGING: Plastic bag and cardboard box.

This PPE's woven materials nor attached components are not known to cause any irritation or inconvenience to the user.

FEATURES:

- adjustable nasal stiffener
- stiffened foldable mask body
- exhale valve (Saana 302 and Saana 304)

DONNING, FITTING AND ENSURING TIGHTNESS OF Saana 301,302,303 and 304 (See Figure 1.)

1. Wash your hands before touching the mask.
2. Take the mask out of the package and open it by pulling the upper and lower panels separately.
3. Place the mask on your face open side in and hold the placement with one hand.
4. Stretch the elastic bands behind your head one at a time and find the best fit.
5. Adjust the mask in a position where it feels most comfortable. adjust the nasal seal to the best position.
6. Before starting to work, make sure that the mask is tight to your face: cover the outside of the mask with your hands and inhale vigorously. If air is felt to flow under the edges of the respirator, adjust the nasal seal and correct the position of the mask until you feel the mask is tight. If you don't get the mask to seal on your face, do not go into a dangerous area. Do not use the mask if you have facial hair as the mask's effectiveness will reduce significantly.

DONNING, FITTING AND ENSURING TIGHTNESS OF Saana City (See Figure 2.)

1. Wash your hands before touching the mask.
2. Take the mask out of the package and open it by pulling the upper and lower panels separately.
- 3b. Place the mask on your face open side in and hold the placement with one hand.
- 4b. Stretch the ear loops around your ears in a way that they are not twisted.
5. Adjust the mask in a position where it feels most comfortable. adjust the nasal seal to the best position.
6. Before starting to work, make sure that the mask is tight to your face: cover the outside of the mask with your hands and inhale vigorously. If air is felt to flow under the edges of the respirator, adjust the nasal seal and correct the position of the mask until you feel the mask is tight. If you don't get the mask to seal on your face, do not go into a dangerous area. Do not use the mask if you have facial hair as the mask's effectiveness will reduce significantly.

The filtering half mask does not protect:

- against gases or vapours
- by lack of oxygen (oxygen must be at least 19.5% by volume)
- by excessive oxygen content
- in situations where the impurities or the risk they can cause are unknown
- if it is not properly sealed by user

Leave your working area and go to safe area immediately if:

- you have difficulties to breathe
- you feel dizzy or fatigue

The user of the filtering half-mask must be instructed in, how to put on the mask, how to ensure it's tightness, how to remove the mask and how to dispose it. Proper use of the mask at all stages of use is important. If the instructions are not followed, the user of the mask may be exposed to health hazards from which the mask is intended to be protecting.

INSPECTION BEFORE USE:

Wash or disinfect your hands before touching the mask. Check that the packaging of the mask is undamaged, that the mask has all its components such as the cup, rubber bands and optional valve and that they are intact. Check the expiration date of the mask and do not use if expired.

USE, REMOVAL AND DISPOSAL OF RESPIRATOR:

Use the mask continuously. Do not touch the outside of the mask during use. Do not temporarily remove it from the face during use. Mask needs to be changed to a new one if you need to remove it from your face or when damaged. Use the mask for a maximum of one working shift. It is recommended that the operating period does not exceed four hours.

Wash or disinfect your hands before removing the mask. Remove without touching the outside of the cup. Lift it off your face, for example by grasping the sides of the respirator. After the PPE has been removed, place it in a rubbish bin provided for this purpose. Finally wash or disinfect your hands.

MAINTENANCE, CLEANING AND DISINFECTION

Disposable non reusable.

STORAGE:

Store the respirators in a dry place with a temperature between -20 and +40 and a relative humidity below 80%

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